

INSTRUCTIONS

GARGLE TEST



1. Create an account

Create your own personal account. If you wish, also add family members or friends. To collect the tests, please bring your smartphone with you. Accounts can also be linked. Find details in the account settings.



Registration:
Scan QR code
or visit:
my.novatium.at



2. Collect & confirm test

Show the staff at the partner branch your pick-up confirmation on your smartphone. Confirm your pick-up. Test kits may only be handed out on display of the Confirmation.

Partner branches:



3. Gargle & scan

Follow the 6 steps below:



1 30 minutes before giving the sample, rinse your mouth with water.



2 Do not eat or drink anything afterwards. Fill the vial half-way with water and gargle with this water 30 seconds.



3 Screw the enclosed funnel onto the vial.



4 Spit the gargled water back into the vial. Dispose the funnel and close the vial tightly.



5 Stick the enclosed QR code on the side of the vial and scan it with "Redeem test" in your Novatium-Account.



6 Now put the tube into the return bag and close it carefully.



4. Submit gargle test

Hand in your test at a return station at one of the partner branches for analysis. The tests are delivered to the laboratory in the course of the day and analysed.



5. Receive your PCR test results

You will receive the laboratory results by E-Mail. You can also view the test results in your account at any time. In order to protect your data, the E-Mail will contain a link to the test certificate (green passport) and the first and last name of the person tested only.



For details please see:
alpbachtal.at/sicherheitskonzept

Alpbachtal Tourismus
Zentrum 1, 6233 Kramsach | +43 5337 21200
info@alpbachtal.at | alpbachtal.at

